


Instructions for replacing a full Daniels Reusable Sharps Container

If your department has been issued a Daniels key (contact department lead for container key)

- 1 Retrieve empty container from Clean Utility Closet
- 2 Permanently lock the full container by following the instructions on the right
- 3 Insert key and unlock bracket so the full container can be removed
- 4 Slide full container off the bracket, and insert new container onto the locking bracket
- 5 Secure and lock new container to the bracket using the Daniels key
- 6 Take full "locked" container to the Soiled Utility Closet for secure storage

If your department has not been issued a Daniels key, make note of the room number where there is a full container and contact dispatch for hospital.

HOW TO LOCK A FULL DANIELS SHARPS CONTAINER

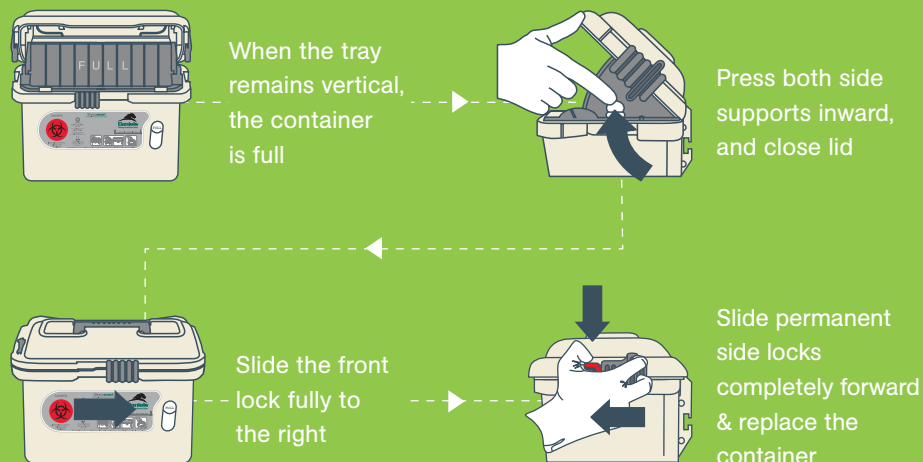

Sharps are discarded objects or devices capable of cutting or penetrating the skin, including the following:



- ▶ Hypodermic needles
- ▶ Intravenous sets
- ▶ Pasteur Pipettes
- ▶ Broken Glass
- ▶ Scalpel blades
- ▶ Various hard plastic items i.e intact amniotic membrane perforators, broken plastic pipettes

The following items should NOT be disposed of in this container:



- ▶ **GARBAGE**
- ▶ **GLOVES**
- ▶ **NON-SHARPS**
- ▶ **PACKAGING**

Each year, Sharpsmart Reusable Collectors avert approximately **250,000 pounds** of plastic & cardboard waste from landfill.

By **choosing reusable**, you are dramatically reducing the impact on our environment.