How To Use Your Secure a Drug



UNDER DIRECT OBSERVATION OF A WITNESS:





Remove medication from packaging: vial, bubble pack or foil

2



Load unused or unwanted controlled medications into the Secure a Drug container

Gently agitate container so the liquid slurry pours over the medications

When the container is full, tighten cap securely to the left aligning the triangle shape on the lid to the "locked" position. Safely dispose of locked container into a non-hazardous or hazardous waste bin.

CORRECT DISPOSAL

Full, used Secure a Drug containers will be returned to Pharmacy for storage until the contracted vendor collects and disposes of the waste.

If Hazardous controlled substances are disposed of into Secure a Drug, the full containers MUST be placed into a RCRA Hazardous container.

SAD211007

PLEASE NOTE

Empty vials and ampules must go into the sharps container. DO NOT place sharps or syringes into Secure a Drug containers. Never dispose of Secure a Drug containers in general trash or RMW bins.





WWW.SECUREADRUG.COM