

## Secure a Sharp Proper Lid Closure



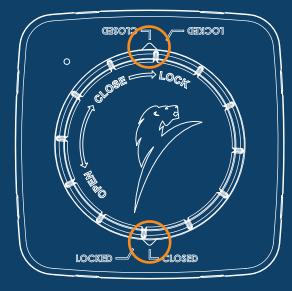

## **CLOSED POSITION**

In the closed position, the lid is sealed and puncture-proof. The cap should not feel loose at this point, if the cap feels loose, twist it further.

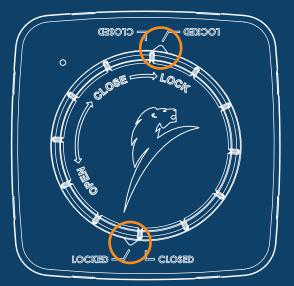

## LOCKED POSITION

In the locked position, the lid is sealed, puncture-proof and will not open during transport. The cap should be twisted to the farthest position to securely lock the container.

LOCKED - L CLOSED

S E CULARD RE

03507.2

The molded arrows on the cap should align with the markings on the container lid to indicate closed or locked position. OCKED -