



# Sharpsmart

REGULAR | INSTRUCTIONS FOR USE

## OPEN

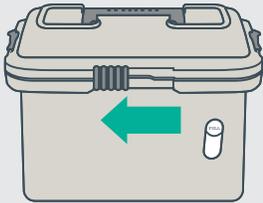


- 1 Slide the front lock to the left.

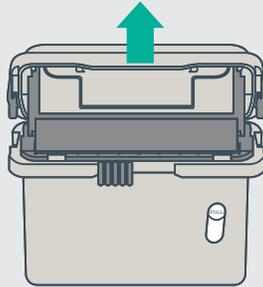


- 2 Lift lid until it locks open.

## SYRINGES

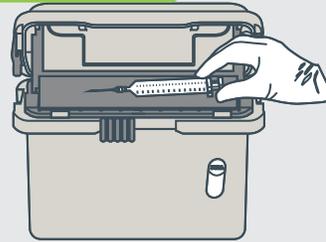


- 1 Drop syringes horizontally onto tray.

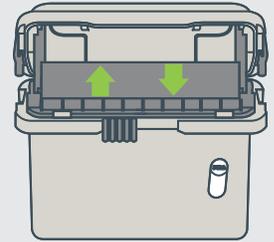


- 2 Tray will empty itself.

## LIGHT SHARPS

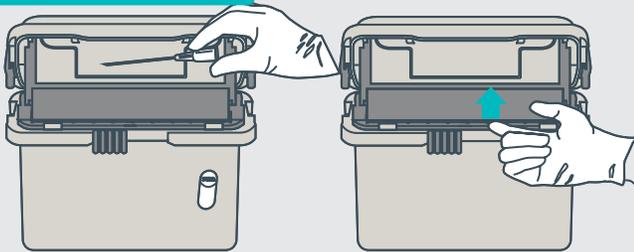

- 1 Drop sharp horizontally onto tray.
- 2 Manually lift the tray to empty.

## FULL

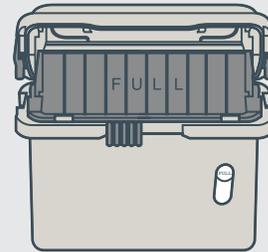

When the tray remains vertical, lock & replace the collector.

## CLOSE



- 1 Press both side supports inward, close lid.



- 2 Slide the front lock fully to the right.

## LOCK



- 1 Depress red tabs.

- 2 Slide locks completely forward.

**PLEASE NOTE**  
Side locks are permanent!

### Remember

Never put your hands into a sharps collector.  
Always close the collector before moving or removing container.  
Do not deface or write on the collector.  
Always dispose of sharps and sharps safety devices into sharps collectors.