

Infection Control

IN YOUR SKILLED NURSING FACILITY



Proper waste management will help protect you, your staff and your residents. Here are five key tips on Infection Control:



1

Fit-for-Purpose and non-contact bins for the collection of infectious waste



2

Following OSHA guidelines for discarding regulated medical-waste items



3

Employing waste handling practices that minimize the “touches” of infectious waste and its outer packaging



4

Tamper-proof containment before collection, ensuring waste is sealed in leakproof packaging



5


Waste Consolidation and readiness for storage and transport that minimizes risk of spillage and cross-contamination




WWW.DANIELSHEALTH.COM | 888 952 5580

PROPER SHARPS DISPOSAL

OPEN

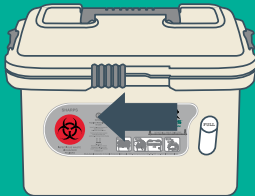


1 Slide the front lock to the left.

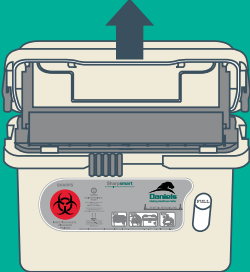


2 Lift lid until it locks open.

SYRINGE DISPOSAL

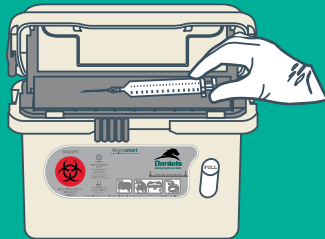


1 Drop syringes horizontally onto tray.

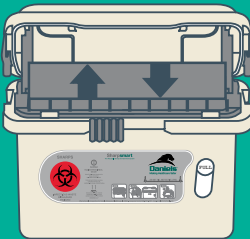


2 Tray will empty itself.

LIGHT SHARPS DISPOSAL

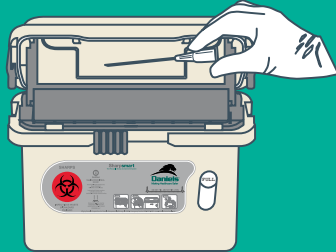


1 Drop syringes horizontally onto tray.

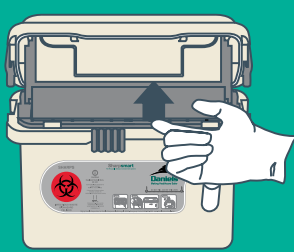


2 Manually lift the tray to empty.

FULL

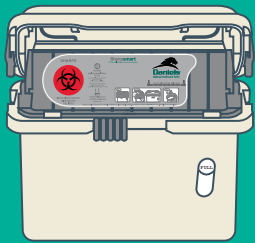


When the tray remains vertical, the collector is full.

CLOSE

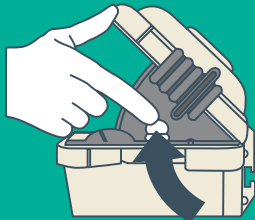


1 Press both side supports inward, and close lid.

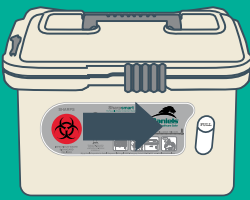

2 Slide the front lock fully to the right.

LOCK

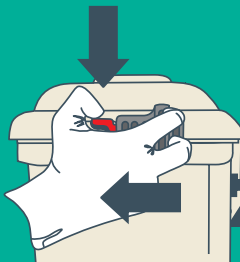

1 Depress red tabs.

2 Slide locks completely forward & replace the collector.

NOTE: SIDE LOCKS ARE PERMANENT!