

# Handling & Filling Reusable Medical Waste Containers



**1**

Open lid of reusable medical waste container.



**2**

Line with an approved Department of Transportation red bag liner.



**3**

Fill with medical waste.



**4**

Grab liner and seal by twisting and hand-tying into a single knot.



**5**

Close lid securely and store in waste storage area ready for collection.



**6**

Once securely closed, add shipping label (hangtag or adhesive) onto the side of the container.

DISCLAIMER: This is a GUIDE ONLY and is intended to provide general information. Users of this guide are strongly encouraged to check applicable local, state and/or federal regulations.